


## How to connect to Smart Phone.

- **JABES Electronic Stethoscope can connect to Android Smart phone only. If you want to connect with other devices, you have to test by yourself firstly.**

1. Put the stereo audio 2m cable(3.5 $\phi$ ) in stethoscope's box to smart phone audio cable's MIC side (Red color).

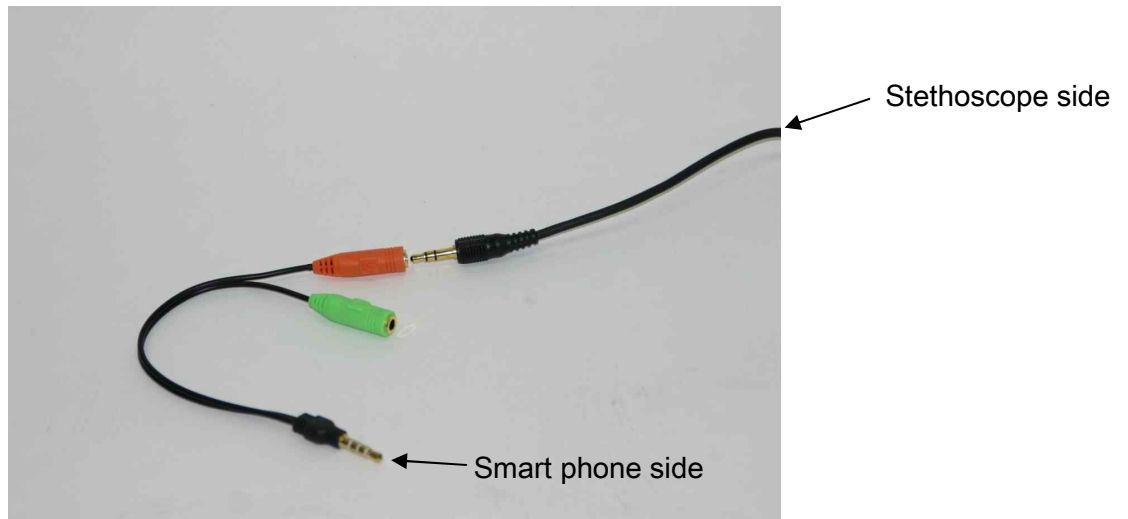


2. Put a ear phone to smart phone audio cable's Green color side (Ear phone, Head phone).



JABES creates Life-Saving products.

- Put the smart phone cable to smart phone's audio plug (4 Pole) and Run Application in the smart phone like Record Application.

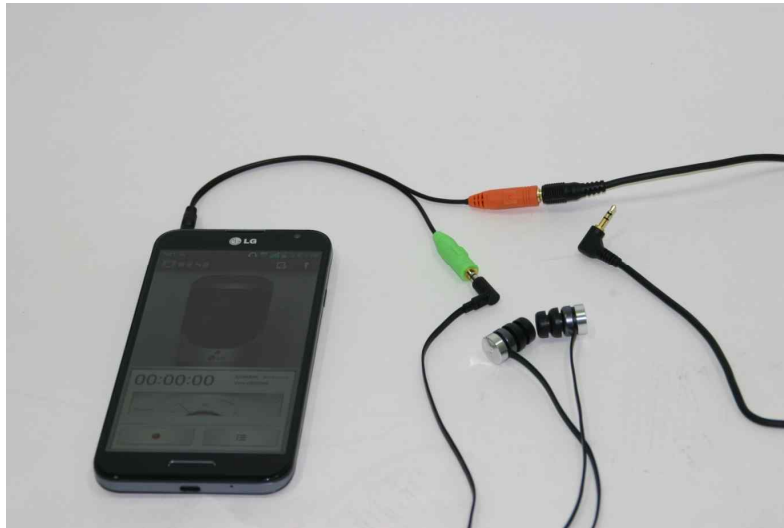

- If you call to other person's smart phone number, you can send your stethoscope sound to the person under 3G communication. The person can listen your body sound even your voice thru his phone using ear phone. Because phone's speaker quality is not fit for heart sound.
- If you use video conference program like skype, you can make tele video diagnosis. Refer to below figure.



JABES creates Life-Saving products.